ThevoTwist · ThevoTherapy

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Headrest height reducer

Item code: 6569

The headrest height reducer can be used as an option, if the upper spring element of the backrest of ThevoTwist or ThevoTherapy has been removed to achieve a lower back height.

Loosen and remove both wing nuts on the right of the headrest adapter. (Fig. 1)

Remove both washers. (Fig. 2)

Now, remove the short metal plate from the right grid joint. Pull the screws out of the grid joints on the left. (Fig. 3)

Now, remove the short metal plate from the left grid joint. (Fig. 4)

Now, mount the headrest height reducer to the left of the headrest adapter (Fig. 5a):

Insert the screw in the upper hole of the headrest height reducer. (Fig. 5b)

Now insert the screw in one grid joint, making sure the headrest height reducer rests in the designated slot. (Fig. 5b)

Add another grid joint, so both grids gear into each other. Now, insert the second screw in the bottom hole of the headrest adapter. Mount the left grid joint as described above.

Now, insert the screw with the upper grid joint in the adapter of the headrest and the screw with the bottom grid joint in the headrest tube. (Fig. 5b)

Important: The locking pin at the headrest tube needs to be on the *right* side. (See mark, fig.3)

The grid joints on the inside have to be positioned, so the plates at the headrest adapter and at the headrest tubes rest in the designated slots. (Fig. 6)

Now attach two grid joints to the top thread on the right, so the grids gear into each other and the plate at the headrest adapter rests in the designated slot of the inside grid.

Mount the bottom grid joint on the right as described above. Insert the headrest height reducer in the designated slots of the grid joints on the right. Reattach the washers. Fasten the wing nuts. (Fig. 7)

To remove the upper spring element in the back of a ThevoTwist or ThevoTherapy seat unit, loosen the Allen screw on the rear and take the spring element off forwards. (Fig. 8)

Press the locking pin on the right of the headrest tube and insert the headrest into the receivers at the back of the ThevoTwist or ThevoTherapy seat unit. (Fig. 9)

Adjust the height desired and lock the headrest with the two knurled nuts provided. (Fig. 10)