A chassis / Swiftly

Crash-Ösen Set

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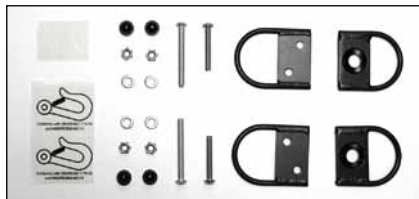
Tie-down kit

Assembly instructions – Accessory

A chassis

1

2

3

4

Swiftly

5

6

7

8

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Tie-down kit – A chassis

item code: 9700 67839 0000

Components Tie-down kit for A chassis (fig. 1)

Rear assembly:

The A chassis is already prepared for assembling the tie-down kit above the rear wheels: Remove the plugs from the predrilled holes (2 holes at each side). Tighten the large u-shaped loops left and right outside to the frame. Attach enclosed black stickers below the loops. (fig. 2)

Front assembly:

Dismount the front wheel incl. fork on each side of the chassis. (fig. 3) Insert the small u-shaped loops. The loops must point diagonally toward driving direction. No further screws are needed for attaching the front loops. Remount the forks and the front wheels. Attach enclosed white stickers near the loops. (fig. 4)

Tie-down kit – Swiftly

item code: 9800 06830 0000

Components Tie-down kit for Swiftly (fig 5)

Rear assembly:

The Swiftly chassis is already prepared for assembling the tie-down kit above the rear wheels: Remove the plugs from the predrilled holes (2 holes at each side). Tighten the large u-shaped loops left and right inside to the frame. Attach enclosed black stickers above the loops. (fig. 6)

Front assembly:

Dismount the front wheel incl. fork on each side of the chassis. (fig. 7) Insert the small u-shaped loops. The loops must point directly toward driving direction. No further screws are needed for attaching the front loops. Remount the forks and the front wheels. Attach enclosed white stickers near the loops. (fig. 8)

General information

The A chassis and Swiftly stroller are designed to be forward-facing when used as a seat in a motor vehicle. However, the occupant should be transferred to the vehicle seat whenever it is feasible. Anyway, the occupant and the stroller must be secured in a vehicle using the vehicle-installed restraint system and a 4-point tie down system.

Please find further important safety information concerning transport in a motor vehicle in the user manuals of the A chassis and Swiftly.

Download current user manual at www.thomashilfen.com