ThevoTwist · ThevoTherapy

- (DE)** **Montageanleitung Zubehör**
Thoraxpelotten, flexibel mit Brustgurt
- (GB)** **Assembly instructions - Accessory**
Lateral trunk supports, flexible with chest belt
- (FR)** **Notice de montage - Accessoires**
Cales-tronc latéraux, flexibles avec sangle thoracique
- (ES)** **Instrucciones de montaje - Accesorios**
Soporte torácico, flexible con cinturón de pecho
- (IT)** **Istruzioni di montaggio - Accessori**
Supporti del busto, flessibili con cinghia pettorale
- (NL)** **Montage handleiding - Accessoires**
Thoraxpelotten, flexibel met borstfixatie
- (NO)** **Monteringsanvisning - Tilbehør**
Sidestøtter, fleksible med brystbelte
- (SE)** **Monteringsanvisning - Tillbehör**
Sidokanalsskydd, mjukt med bröstsele
- (DK)** **Montage-vejledning - Tilbehør**
Thoraxpelletter, fleksibel med brystbælte
- (PL)** **Instrukcja montażu - osprzęt**
Poduszki na klatkę piersiową, elastyczne z pasem piersiowym

1

2

DE

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GB

ThevoTwist Lateral trunk supports, flexible with chest belt

Item code: 6596

ThevoTherapy Lateral trunk supports, flexible with chest belt

Item code: 8427

Insert the lateral trunk supports at suitable chest height into the slots of the back cushion or the open zip. Into the openings in the back pad at suitable chest height, or open the zip. The clamp is fastened both flexible fibre glass tracks. Before assembly, the external wing suspensions on the two flexible fibre glass tracks must be removed. Now put the clamp onto the flexible fibre glass tracks and fasten with the screws. The chest pads can individually be adjusted in width (1). The pad can also be adjusted in height, in 15 mm/ 0.6" steps. To do so, remove the Allen screw, set the desired height and fasten the screw again (2).

You can adapt the supports individually by carefully bending them. Close the chest belt with the buckle. The belt length is individually adjustable on both sides.

Note: By attaching the lateral trunk supports in the rotational movement of the spring element may be restricted.

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