

ThevoActiv

Ⓧ	DE	Bedienerhandbuch – ThevoActiv	6
Ⓧ	GB	User manual – ThevoActiv	12
Ⓧ	NL	Gebruiksaanwijzing – ThevoActiv	19
Ⓧ	DK	Brugermanual – ThevoActiv	26
Ⓧ	PL	Instrukcja obsługi – ThevoautoActiv	32
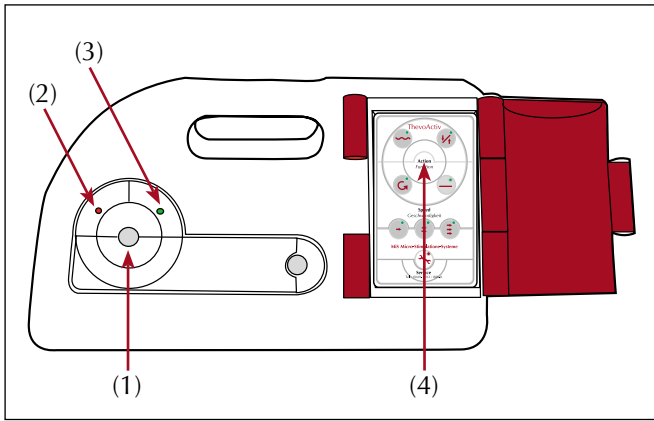


(1)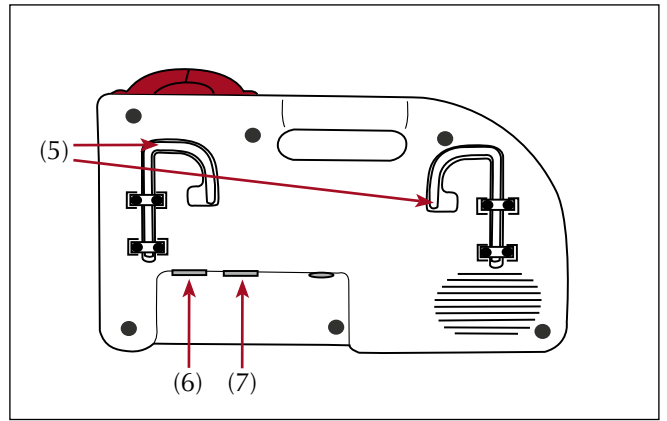


(2)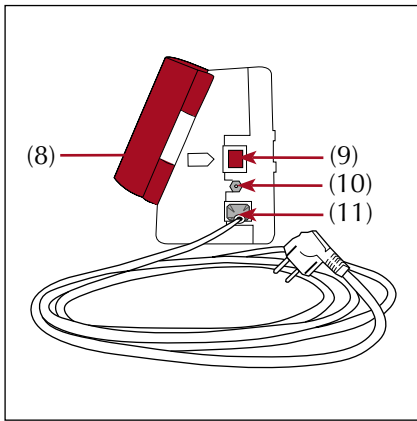


(3)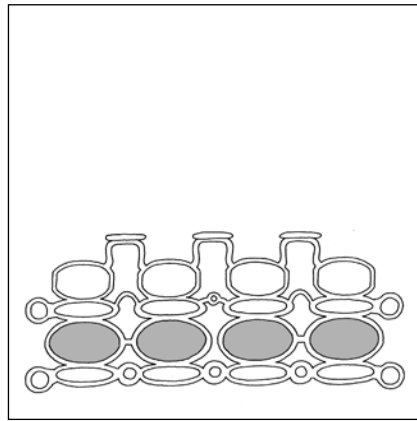


(4)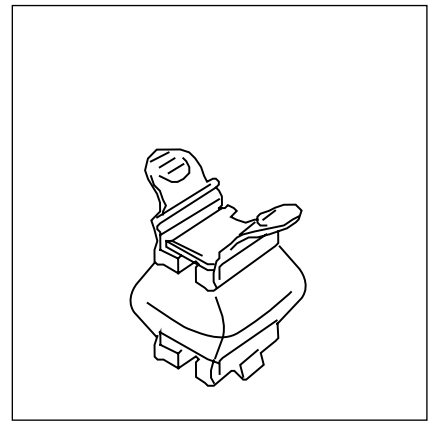


(5)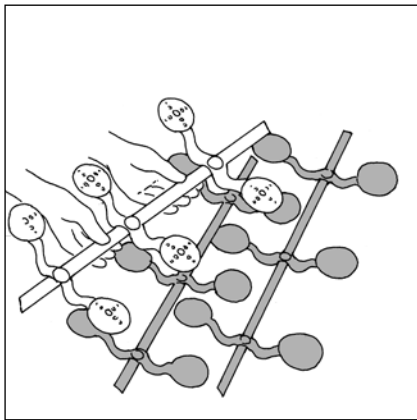


(6)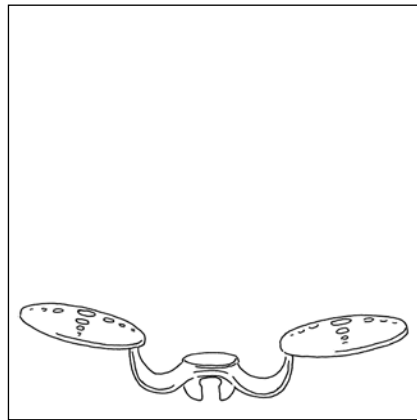


(7)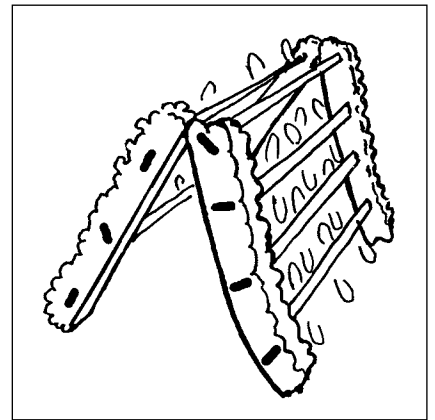


(8)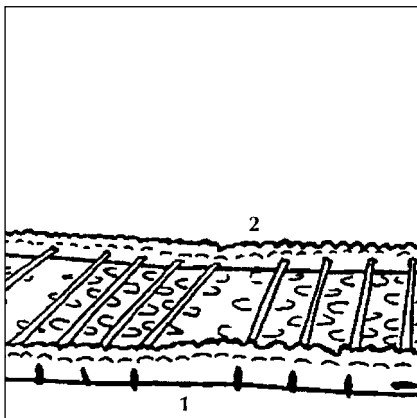


(9)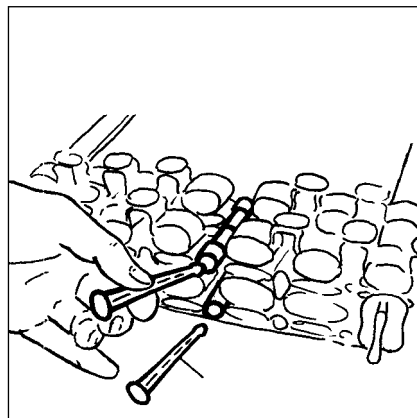


(10)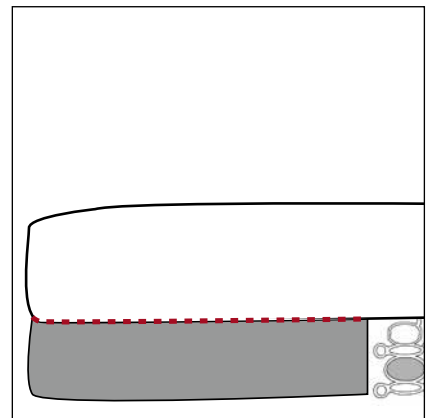


(11)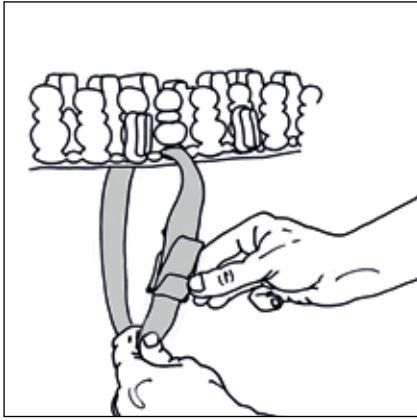


(12)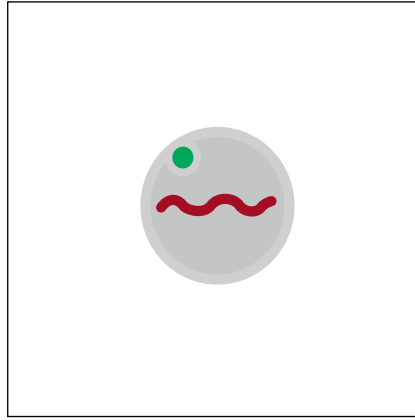


(13)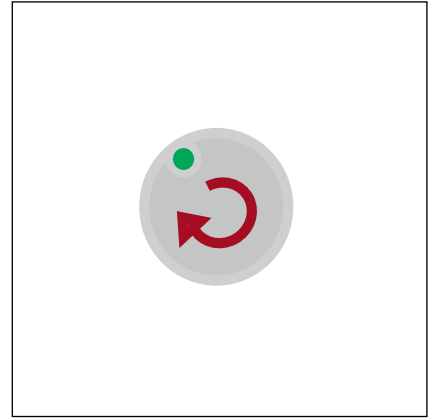

(14)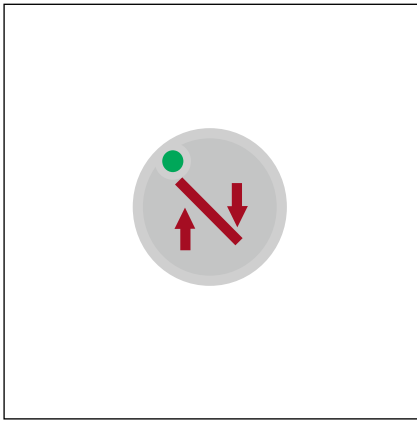

(15)

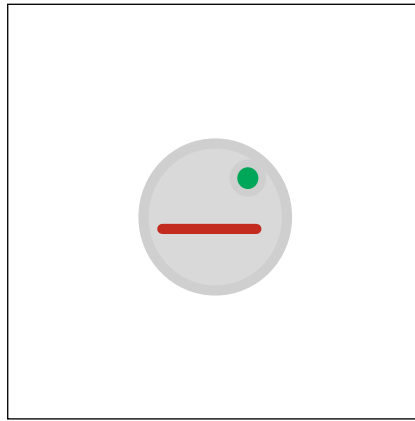(16)

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Dear customer,

with ThevoActiv you have purchased a mobility bed, which meets the highest quality demands of ambulant and stationary care. For ThevoActiv we have used high-quality materials, to ensure long durability and low maintenance

Before using ThevoActiv we recommend that you read this user manual carefully. It contains a lot of helpful information and hints. To achieve a best possible therapeutic outcome, the system needs to be adjusted to the individual patient.

If you have any questions, problems or comments, please contact your local dealer or us directly:

Please visit our website for further information on the prevention and treatment of pressure sores at **www.thevo.info**

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Contents

	Page
Prevention and treatment of pressure sores	13
Components	13
Before starting up / assembling the system	13
Start-up of ThevoActiv.....	14
Changing the movement pattern	14
The movement patterns of ThevoActiv	14
Additional valves.....	15
Disinfection methods and cleaning recommendations	15
Maintenance	15
Indications/contraindications	16
Self test / service LED	16
Conditions of warranty	16
Safety hints	17
Disposal of the system.....	17
Technical data	17

Prevention and treatment of pressure sores

ThevoActiv has been developed for the prevention and treatment of pressure sores **up to and including grade IV according to EPUAP** (European Pressure Ulcer Advisory Panel).

ThevoActiv is particularly suited for the care of patients with neurological clinical pictures and pain symptoms. Besides an excellent pressure distribution, the system is well suited for the perception and movement support.

Components

Control unit and keypad

ThevoActiv is operated by a control unit. The integrated microprocessor controls the movement of the actors. The control unit can be hung up to the bed with two fixation clips.

Via the keypad different movement patterns can be set according to the demands of the patient. This ensures individual prevention and treatment of pressure sores and support of body perception.

Control unit, front (img. 1)

Start / Stop-switch (1), Failure indication (2), Status of operation (3), keypad (4)

Control unit, back (img. 2)

Fixation clip (5), Port left bed side (6), Port right bed side (7)

Control unit, sideways (img. 3)

Key pad (8), Power switch (9), Pump port (10), Power plug (11)

Active side element

The active side element does not only connect the single flexible fibreglass-tracks. It also offers the possibility to integrate ThevoActiv into common nursing and hospital beds. The system is simply laid on the existing slatted frame or inlay frame. Due to the flexible construction you can still use the head and foot adjustment as well as the height adjustment of the bed.

Valves

The valves are small bellows integrated in the active side elements. Individually adjustable movement patterns are achieved by feeding air through the supply pipe which slightly lifts the flexible fibreglass-tracks. This results in a relief of pressure in the vulnerable areas.

Flexible fibreglass-track

These tracks are made from durable material by innovative glass fibre technology (pultrusion). In addition to a long durability, they ensure optimal lying comfort, which has a favourable effect on the recreation of the patient.

Wing suspension

3 or 4 wing suspensions situated on one flexible fibreglass-track optimally adapt to the body contours of the patient. An anatomically correct body alignment is ensured and sleep is influenced positively. Each wing suspension can be moved or removed in order to support the individual preventive and therapeutic treatment in an optimum way.

Before starting up / assembling the system

Before starting up the ThevoActiv mobility bed, please follow below stated worksteps:

1. Plug the 2 cable harnesses of the system into the sockets 6 and 7 and on the back of the control unit. Right bed side (img. 2, 6), left bed side (img. 2, 7).
2. Plug the air hose into the pump port (img. 3, 10) sideways at the control unit.
3. Plug the power plug in (img. 3, 11).

ThevoActiv is now ready for operation and for being used for the prevention and treatment of pressure ulcers. The start-up of the different functions will be described in the following chapters. Please read them carefully to ensure optimum prevention or treatment and to avoid failures.

Start-up of ThevoActiv

a) Assembly

1. Please note, that the ThevoActiv mattress is delivered in an extra box. The underframe is folded in the middle (img. 8) and equipped with a cover. Lay the underframe flat on the floor, ensure that the side elements interlock (img. 9) and open the cover. The connection thorns have to be placed tight and flush into the side elements at the connection points 1 and 2. (img 10) Close the cover afterwards.
2. Lay the underframe on the existing slatted frame or inlay frame of your bed. Please pay attention to the label for the head and foot section.
3. Now lay the mattress on the underframe.

Important hint: ThevoActiv can also be used without a pre-existing slatted frame. In this case the supporting surfaces along the sides of the bed base must be at least 10 cm (3,9 ") wide.

Hint: If you have a bed with side rails you will probably need to use side rail extensions.

4. Pull the supply pipe through one of the holes in the black fixation flaps of the mattress cover.
5. The black fixation flaps of the mattress cover in the head and foot section must be flapped around the underframe and closed with the zipper. (img. 11)
6. The underframe of ThevoActiv is safely fixed to the bed frame with the 6 belts. Thread the belts around the tubular frame of the bed. Thread the belt through the buckle and pull it tight. To release the belt, open the buckle and pull out the belt. (img. 12)

Attention: If you have a height adjustable bed, please fix the belts to the lifting system.

b) Securely fix the control unit to the bed.

c) Plug the line cord of the control unit (230V or 110 V) into a power socket.

d) Switch the power switch on(light turns red).

Changing the movement pattern

As soon as the light emitting diode (LED) on the front of the device flashes, please press the start button at the control unit and the system will start working. The last chosen program starts running, indicated by the LEDs at the control keys (img. 1, 4).

- a) Stop the currently running active movement pattern by pressing the start/stop-switch at the control unit.
- b) Select the now desired new movement pattern (wave, rotation, inclined plane, static) and/or its speed level (fast, medium, slow) by tapping on the corresponding function key. The selection will be confirmed by flashing of the corresponding LED. The function „static“ (bar symbol) is without speed.
- c) Re-activate the system by pressing the start/stop-switch.

Hint: In case of an emergency (resuscitation), you can stop the movement pattern by pressing the power switch!

The movement patterns of ThevoActiv

Due to different movement patterns ThevoActiv can be adjusted to the patient's individual needs. The system runs in an interval of 6 minutes movement pattern and 4 minutes pause. The pause is indicated by flashing of the green "Activ" LED.

The following movement patterns are available:

1. Wave (img. 13)

The actors are approached parallelly right and left. The stimulation pattern is similar to a wave that occasionally flows from the feet to the head and back. The speed can be selected, as desired, in three steps from slow to fast. This stimulation pattern allows a body perception of the patient from head to foot.

2. Rotation (img. 14)

The actors are approached staggered right and left. This results in reverse wave mountains being globally looked upon as equivalent to a rotation. The direction of rotation is reversed after 3 runs. Here too, the flow speed can be chosen in three steps. The motion of rotation ensures an increased attention with the patient. Due to the stimulation of body zones locally lying far apart from each other and the reverse direction a perception of the body is given in its length, but also in its right and left body half.

3. Inclined plane (img 15)

All actors of one plane are approached simultaneously and thus create an inclined plane. After a previously set time allowance the inclined plane shifts to the other side. With this stimulation pattern the right and left body halves are perceived separately and especially the patient's vestibular sense is promoted additionally.

4. Static (img. 16)

This function does not perform any movement pattern. In the interval all actors are evenly lifted.

The throughput speeds can be adjusted, as well:

Fast: 26 sec. Medium: 78 sec. Slow: 120 sec.

Additional valves

As standard ThevoActiv is equipped with 16 valves, carrying out the selected movement pattern. This provides a good stimulation of the entire body surface. However, certain patients may require a more sensible, fine adjusted stimulation to support body perception and mobility. Here you can use a ThevoActiv system with 24 valves. Please contact us for further information.

Disinfection methods and cleaning recommendations

Mattress

The covers can easily be taken off due to a sewed on two-sided zipper. The mattresses are autoclavable and therefore suitable for reuse.

Jersey cover

- washable up to 65 °C (149 °F) • suitable for tumble dryers • breathable

Incontinence cover

- washable up to 65 °C (149 °F) • suitable for tumble dryers • to be cleaned by wiping disinfection

Under frame

Easy cleaning: the system can be cleaned with non-corrosive detergent. Please ensure the system to be dried carefully after every cleaning. The system can also be cleaned with disinfectant on alcohol basis.

Disinfection

For the disinfection we recommend two methods accepted by the Robert-Koch-Institute:

- the fractionated vacuum method (VDV-method) up to a max. temperature of 110 °C (230 °F).
- the steam flow method up to a temperature of 110 °C (230 °F).

Caution: The control unit must not undergo a steam sterilisation! It can be wiped with a damp cloth, but should in no case be exposed to an open jet of water.

Hint: A thermal treatment over 110° C (230 °F) causes premature ageing of the plastic parts and should therefore be avoided. Please make sure when, using chemical disinfectants, that they are suitable for plastic.

Maintenance

In order to ensure a long life cycle and utilisability of the mobility bed ThevoActiv, you should follow below stated maintenance instructions:

- Should the red LED „Wrench Symbol“ glow permanently, please call a service technician. Nevertheless the system can be used without any loss of functioning.
- We recommend an annual inspection carried out by qualified personnel.
- The tight fit and impermeability of the actors shall be checked regularly. Leakages, leading to punctual losses of function are recognised by a loud fizzling. In such cases, tightly press the valve unit in direction of the bellows, whilst supporting the bellow from the exterior with your thumb.

Important hint: The under frame should be used in combination with the corresponding mattresses only. Please pay attention to the following:

- ThevoActiv 50 up to 50 kg (110.2 lb) body weight
- ThevoActiv 100 from 50 to 100 kg (110.2 to 220.5 lb) body weight
- ThevoActiv 135 from 100 to 135 kg (297.6 lb) body weight

If the body weight is distributed very unequally, (i.e. strong pelvis or stomach, leg amputation) a mattress for higher weight should be used.

Indications / contraindications

Indications

ThevoActiv is suitable for the following clinical pictures:

- Prevention of pressure sores (incl. high-risk patients)
- Pressure ulcer patients up to and incl. grade IV acc. to EPUAP
- Pain patients, e.g.
 - Osteoporosis
 - Rheumatism
 - Soft tissue rheumatism
 - Gout
 - Arthritis
 - Tumour patients
- Perception disorders, e.g. after stroke

Contraindications

The use of ThevoActiv is not recommended for the following clinical pictures:

- Lyell's Syndrome
- Status after rotation flap plastics
- Large area 3rd degree burns at the back

Hint: Referring to the previous practical use and several tests executed, we are not knowledgeable of any problem in the use of this product.

Self test / service LED

Every time the system is switched on (power switch), an automatic self test is carried out, indicating, if applicable, the following errors by the red service LED (wrench-symbol):

- **Continuous flashing**
Not both of the cable harnesses are plugged in.

Note: First plug in the cable harnesses, then press the power switch!

- **Double flash:**
The number of connected valves on the left and right side does not match.
- **Permanent glow:**
Compressor is due for maintenance. Call the service hotline!

Furthermore:

The service LED also flashes temporarily (3 seconds), if you try to change the movement pattern or speed while the system is operating.

Conditions of warranty

Thomashilfen takes full responsibility for the quality of ThevoActiv. All materials as well as the processing have been chosen under the highest production standards. All systems have been tested very carefully before leaving the factory. Thomashilfen offers a guarantee of 24 months.

You should therefore carefully keep all documents on ThevoActiv. The guarantee covers all claims on the basis of production or material failures, which impair functionality. Excepted are wear or tear caused by over or inappropriate use.

Safety hints

Please pay attention to the following points and keep this user manual near to the system, so that you can read it up any time.

- Only qualified and by company Thomashilfen authorised personnel is allowed to open the control unit. Before opening pull the power plug!
- Keep the control unit away from liquids and open flames.
- In case of damages at the housing or at the supply line, firstly pull the power plug.
- Do not store the system at more than 50 °C (122 °F).
- In certain cases medical electrical instruments can be influenced by mobile phones. If there should be a disturbance by a mobile, the system has to be switched off. Switch it on again after 10 seconds. If the problem still exists, please call the service.
- Don't use ThevoActiv in humid rooms.
- ThevoActiv must not be used near to explosive gases.
- In case of damages to the system, do not use it until a repair has been carried out by authorised personnel.

Safety features

Under adverse operating conditions it can happen that the system shuts-off automatically to avoid damage of the device. These safety features apply for short-circuits, disconnection of power and thermal overload.

Short-circuits / interruptions

In case of short-circuits in the actors due to mechanical damage, the short-circuits will be recognised by the system and the wiring concerned will be switched-off. The "Error"-LED at the front of the casing lights up red and the system stops. A re-start is possible by pressing the start button, whereas the defective will be excluded from the information flow, that means this actor is no more functioning. The defective actor has to be exchanged at the first opportunity. The remaining actors and the system remain fully functional. If the defective actor is located close to a "critical caring region", that means directly at the prevention or treatment area, the exchange has to be made as soon as possible.

Thermal overload

In case of high operating temperatures due to unfavourable environmental conditions, as high outside temperature and disturbance of the influx of fresh air, the system switches-off well in advance of damage. A continuing excessive operating temperature inside the system otherwise would lead to early obsolescence. In this case, the operation will be stopped and the red "Error-LED" on the front side of the system will flash. A re-start by pressing the start-button is possible only, if the measured temperature inside the system has fallen below an uncritical threshold value, defined by the manufacturer. The red LED stops flashing and the system is ready for operation again. In case of a repeated occurrence of temperature switching-off, make sure there is a ventilating gap between the back plate of the system and the bed's foot end. Please also make sure, whether this constructively intended gap is not blocked with linen or similar things.

Disposal of the system

ThevoActiv shall not be disposed with the normal household waste. Please turn to registered disposal companies or return the system to your specialist dealer.

Technical data

Complete system

Outer dimensions ThevoActiv	90 x 200 cm/ 35.4 x 78.7 "	Height	100 x 200 cm/ 39.4 x 78.7 "	Height
up to 50 kg (110.2 lbs)	21.8 kg/ 48.1 lbs	19 cm/ 7.5 "	23.6 kg / 52 lbs	19 cm/ 7.5 "
50 to 100 kg (110.2 to 220.5 lbs)	22.8 kg/ 50.3 lbs		24.7 kg / 54.5 lbs	
100 to 135 kg (220.5 to 297.6 lbs)	24.6 kg/ 54.2 lbs		26.7 kg / 58.9 lbs	

Other measurements on request.

Control unit

Operational voltage:	230 V ~ 50 Hz
Max. power consumption:	50 W
Protection class I:	IPX 0
Measurements of control unit:	38 x 21 x 21 cm (14.96 x 8.24 x 5.24")
Medical device:	Class 1 acc. to guideline 93/42/EEC

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