EASyS Advantage · EASyS Advantage S EASyS Modular S · tRide

5-Punkt-Gurt

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5-point belt

Assembly instructions – Accessory

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Notice de montage – Accessoires

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Pull the two upper straps (with the large pads) through the openings in the padding and insert them through the slots in the front back plate. (Fig. 2)
To make installation easier, pull out the back plate slightly. To do this, loosen the two thumbscrews at the back of the backrest.

Then pull the straps into the folding buckles and adjust to the preferred strap length (Fig. 3).

Tip: The strap length of the upper straps can also be adjusted at the front by pulling the D-ring to shorten the strap. (Fig. 4)

The two side strap ends (with the smaller pads) can be attached to the back plate at the rear or to the side of the plastic element with folding buckles. (Fig. 5-6)
Pull the straps to the right and left of the back upholstery to the corresponding folding buckle and fasten them.

To fit the crotch strap, the upholstery must be opened in the stitched seams. (Fig. 7)
The lower belt is passed through the seat cushion. (Fig. 8)

The fleece tape is glued over the slot on the seat plate. (Fig. 9+10)

The thumbscrew is inserted through the eyelet (Fig. 11) and screwed in front of the seat plate. (Fig. 12)

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany