tRide · EASyS · EASyS Modular

(DE)

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GB

H-belt	Item code: 6629		
Butterfly / chest harness	Item code: 6616 (S)	Item code: 6617 (M)	Item code: 6717 (L)
Butterfly / chest harness, flexible	Item code: 6385/2 (S)	Item code: 6585/2 (M)	
Seat waistcoat	Item code: 6631 (S)	Item code: 6632 (M)	Item code: 6731 (L)

Cut the backrest padding in the quilted seam depending on the desired mounting height or open the backrest padding flap. Pull the zip down to open the padding, unhook and release the hook and loop fastener.

Pass the two **upper belts** through the holes in the upholstery and insert them into the front back plate.

To facilitate mounting pull the back plate out slightly. For this, loosen the two knurled screws at the back.

Insert the belts into the cam buckles and adjust the desired belt length (fig. 1). Reclose the backrest padding flap with the hook and loop fastener. Adjust the desired back height and retighten the knurled screws.

The two **bottom belt ends** can be attached with cam buckles to either the back plate or the plastic element at the side (fig. 2-3). Guide the belts left and right next to the back upholstery to the corresponding cam buckle and fix them.

Tip: The belt length can also be adjusted from the front. Pull the D-ring to shorten the belt.

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