EASyS Advantage · EASyS Advantage S EASyS Modular S · Swifty · Swifty 2 · tRide

DE

Sitzhose

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EN

Soft pelvic harness

Assembly instructions – Accessory

FR

Harnais pelvien

Notice de montage – Accessoires

ES

Arnés pélvico

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Place soft pelvic harness with the gray side upwards onto the seat (metal eyelets pointing backwards in direction of travel). (fig. 1)

For size 2, the length of the rear straps can be adjusted using the folding buckles.

Swiftly and Swiftly 2: Open the prefabricated slots in the rear part of the seat cushion and fasten the belts (with metal eyelets).

Swiftly: Loosen the two screws of the rear seat plate and fix the belts with them (fig. 2).

Swiftly 2: Loosen the two soft-touch handles under the seat and fasten the two rear straps (with metal eyelets). (fig. 3)

Other models: Insert the rear straps between the seat and back upholstery and fasten them to the rear screws of the seat plates using the soft touch handles. (fig. 4)

The front straps are guided laterally through the pads or pads and fastened in the appropriate folding buckles. These are located under the seat plate, laterally or on the back plate. (fig. 5)

Finally, close the backrest upholstery again (if available).

TIP: The belt length can also be adjusted from the front. Pull the D-ring to shorten the belt.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany