




VELA Salsa 400

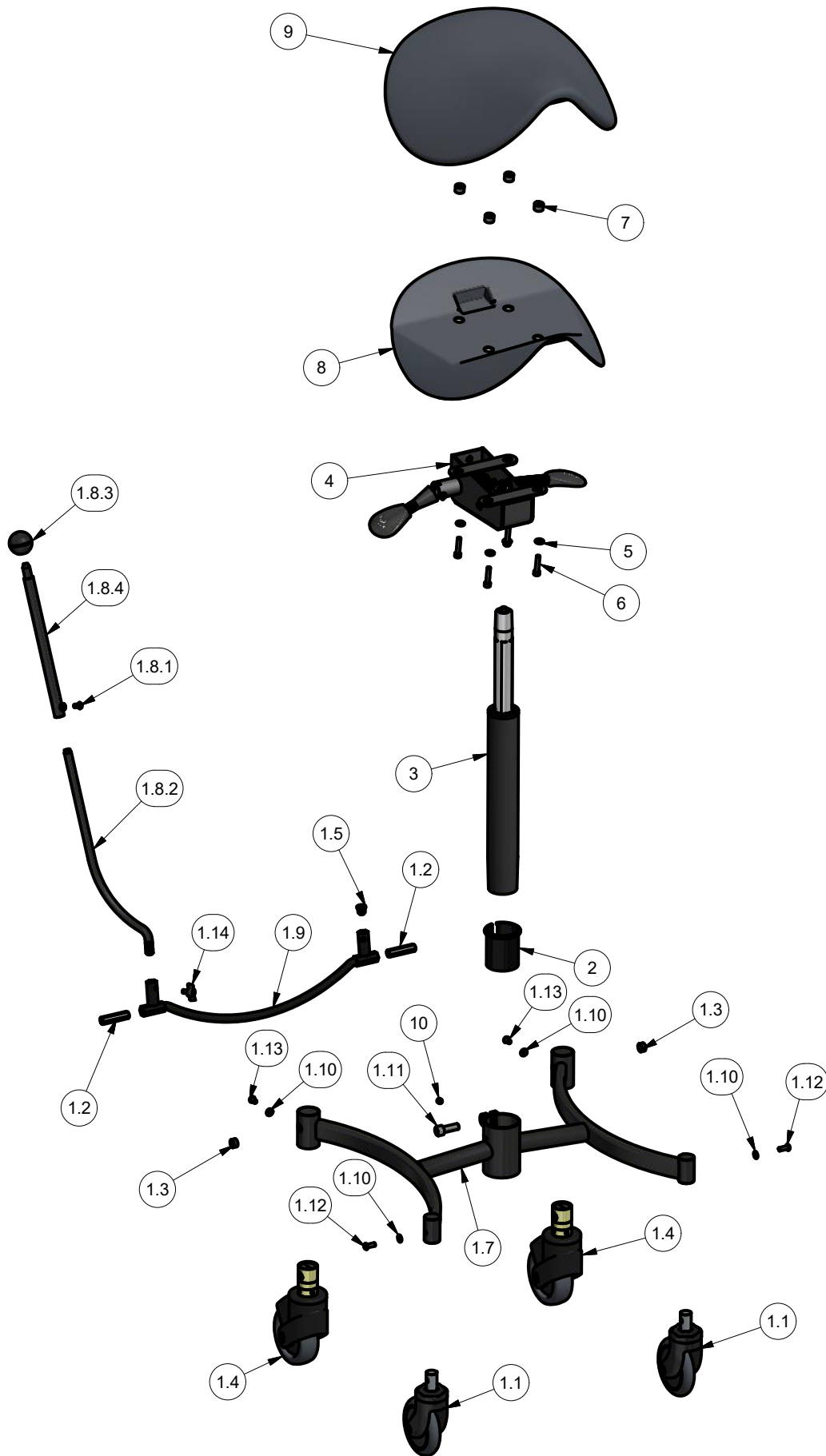

Pos	Pcs	Beskrivelse	Description	Beschreibung	Stock no
1.1	2	Hjul sporkugleleje 100 mm	Castor 100 mm	Rolle 100 mm	014915
1.2	2	Sekskant tap for bremsestang	Hexagon tap brake stick	6-kant Zapfen für Bremsstangen	923451
1.3	2	Rubber plug Ø14x7,5 mm	Rubber plug Ø14x7,5 mm	Gummistopfen Ø14x7,5 mm	923366
1.4	2	Bremsehjul Soft Lock 100 mm	Castor 100 mm	Rolle 100 mm	014015
1.5	1	Rørben Ø15	Plug Ø15 mm	Abdeckstopfen Ø15 mm	921694
1.7	1	Fod H3 sort metallic	Frame H3 black metallic	Untergestell H3 Schwarz metallic	035379946
1.8	1	Forlænger stykke monteret	Extension piece mounted	Verlängerungsstück montiert	926762
1.8.1	1	Skrue med invendig sekskant halvrandt hoved	Hexagon Socket Button Head Screw	Innensechskant Rundkopfschraube	8003806010
1.8.2	1	Bremsestang buet med sekskant sort metallic	Brake lever curved for Flex	Bremsstange gebogen für Flex	925726
1.8.3	1	Håndkugle for bremsestang sort	Ball for Brake Stick, Black	Kugel für Bremsstange, Schwarz	923449
1.8.4	1	Forlængerstykke f/bremsestang sort metallic	Extension bar F/ brake stick black metallic	Verl.stange F/ Bremsstange schwarz metallic	926137
1.9	1	Bremseaksel Flex m/6-kt black metallic	Brake Shaft Uni Flex black metallic	Bremswelle Uni Flex black metallic	925727
1.10	4	Skive	Washer	Scheibe	8802806
1.11	1	Skrue med invendig sekskant cylindrisk hoved	Cylinder Head Cap Screw	Zylinderschrauben mit Innensechskant	8001210030
1.12	2	Skrue med invendig sekskant halvrandt hoved	Hexagon Socket Button Head Screw	Innensechskant Rundkopfschraube	8003806016
1.13	2	Skrue med invendig sekskant halvrandt hoved	Hexagon Socket Button Head Screw	Innensechskant Rundkopfschraube	8003806010
1.14	1	Vingemøtrik M6 x 11	Wing nut M6 x 11	Flügelmutter M6 x 11	927924
2	1	Foring for Ø50 konus/yderrør koksgrå	Conical Busching Ø50 grey	Buchse konisch Ø50 Grau	924928
3	1	KGS gasfjeder 210 mm 300 N	KGS Column 210 mm 300 N	Gasfeder 210 mm 300 N	093030201
4	1	Bæreprade 41.2 mont. m/ betj.greb (Midi u/ ryg) V	Seat support fitted w/handles	Trageplatte m/auslöserhebeln	18411149999930
5	4	Skive	Washer	Scheibe	8802207
6	4	Skrue 6-kt CH M6 x 30 mm	Cylinder Head Cap Screw M6 x 30 mm	Zylinderschrauben m/Innen 6-kant M6x30 mm	8001206030
7	4	Afstandskive plast 15X8X10	Distance washer plastic 20x1,5x4	Distanzscheibe aus Kunststoff 20x1, 5x4	925072
8	1	Plast afdækning for saddelsæde	Plastic cover f/saddle seat	Plastik abdeckung f/Sattlesitz	925035
9	1	Saddel sæde	Saddle seat	Sattelle sitz	06164520671
10	1	Stopskrue med invendig sekskant med krater	Hexagon Socket Set Screw	Gewindestift	8004108008