




VELA Samba 100

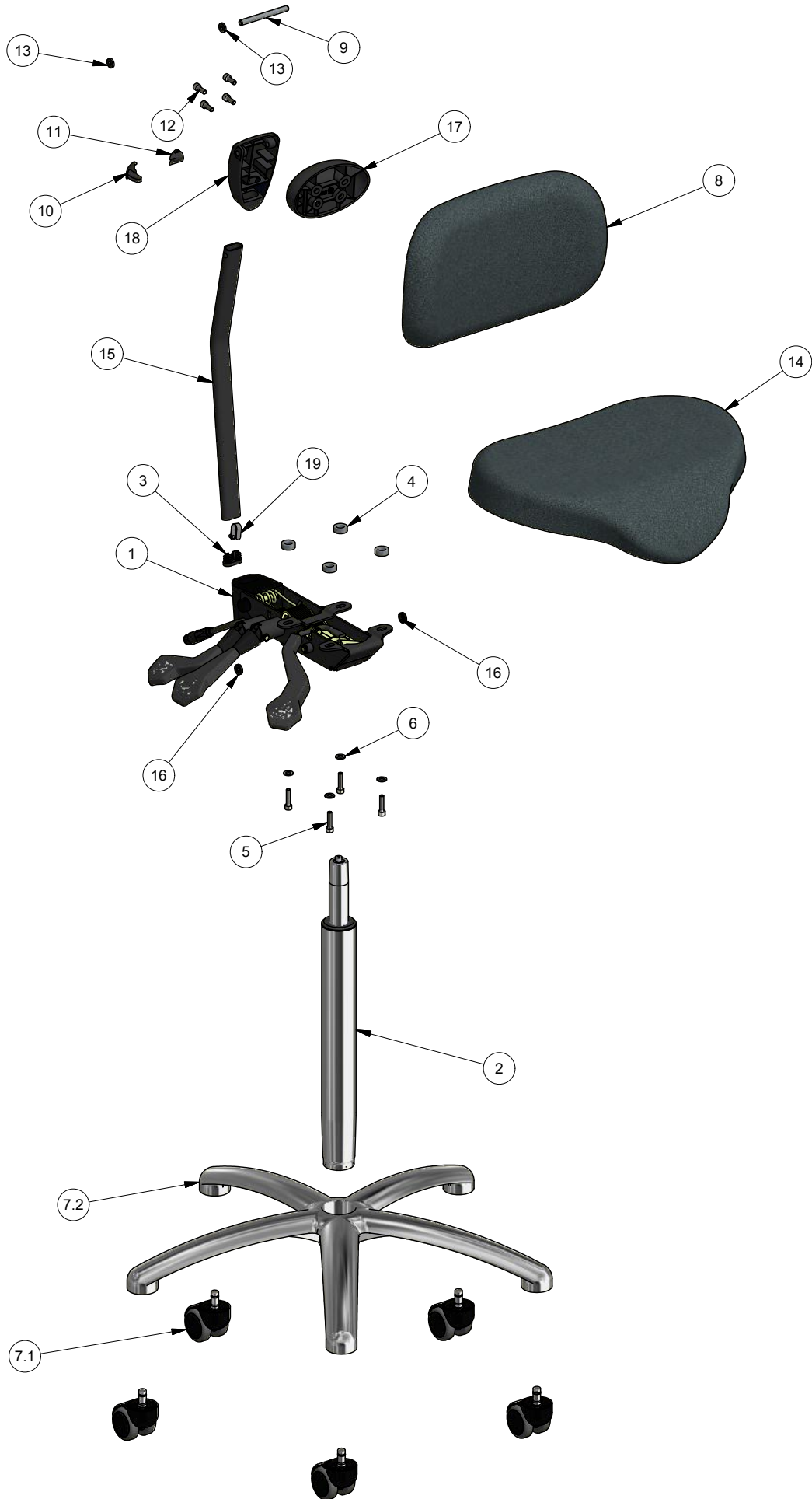

Pos	Pcs	Beskrivelse	Description	Beschreibung	Item no.
1	1	Bæreplade 42.2 mont. m/ betj.greb	Seat support 42.2 mounted w/handles	Sitztrageplatte 42.2 montiert m/Griffe	18422155995555
2	1	Gasfjeder Chrom yderrør, 300 N, 260 mm	Gas spring Chrome, 300 N, 260 mm	Gasfeder chrome, 300 N, 260 mm	091030260
3	1	Rørben 15x30 mm flad-oval m/hak	Plug 15x30 mm	Stöpsel 15x30 mm	925890
4	4	Afstandskive Ø20x8	Distance disc Ø20x8	Distanzscheibe Ø20x8	924886
5	4	Skrue m/indv. 6-kt CH M6x25	Cylinder Head Cap Screw M6x25	Zylinderschrauben m/Innen 6-kt M6x25	8001206025
6	4	Skive	Washer	Scheibe	8802207
7.1	5	Hjul Dobb. blød tap 11x20	Wheel twin soft 11x20	Doppel-Räder Soft 11x20	013202
7.2	1	Alu fod type 3, Blank alu., N/	Underframe type 3, aluminium	Untergestell type 3, aluminium	030329999
8	1	Ergo ryg 38x22 cm, Bore R2	Ergo back rest 38x22 cm, Bore R2	Rücken Ergo 38x22 cm, Bore R2	08015111671
9	1	Aksel for ryghængsel	Shaft for backrest bracket	Oberteil für Rückenlehne Halterung	923284
10	1	Låsetap for ryghængsel	Locking tap for backrest hinge	Schliesstap für Rückenscharnier	926637
11	1	Lås for ryghængsel	Lock for backrest hinge	Schliess für Rückenscharnier	926638
12	4	Skrue m/indv. 6-kt CH	Cylinder Head Cap Screw	Zylinderschrauben m/Innen sechskant	8001206016
13	2	Gummiskive underlag ryghængsel	Rubber disc for backrest hinge	Gummischeibe für Rückenscharnier	926585
14	1	Sæde Delta 46x35 cm Bore 4.2(midi)	Seat Delta 46x35 cm Bore 4 (midi)	Sitz Delta 46x35 cm Bore 4 (midi)	06180712671
15	1	Rygbøjle 21 - 420 mm sort	Backrest arm 21 - 420 mm black	Rückenbügel 21 - 420 mm Schwarz	07333
16	2	Undersænket skive Ø15 x 4	Countersunk washer Ø15 x 4	Senkscheibe Ø15 x 4	926242
17	1	Underlag ryghængsel u/stop	Plate part for Hinge without stop	Aufnahme f/ Scharniere ohne Stopf	926636
18	1	Ryghængsel sort	Backrest hinge black	Rückenscharnier Schwarz	925138
19	1	Knopfeder for 10 mm rør	Pop-out pin for 10 mm tube	Druck-stifte Ø10 mm Rohr	923362